SmartPractice

3400 E McDowell Rd • Phoenix, AZ 85008 • U.S.A.
900.876.3837 • +1.602.225.9090
smartpractice.com/dermatology

Dr. Ebeling & Assoc. GmbH
Issestrasse 5 • 20144 Hamburg • Germany

EC IMPORTER

SmartPractice Europe GmbH
Börsenmonstrasse 8 • 46298 Greven • Germany
+49 40 52470700

Rx Only (for U.S.)

MMAL 3218 © 2020 SmartPractice • All rights reserved
ALUMIN. DIC-0014 R 2

ENGLISH

Finn Chambers on Scanpor

Indications for use:

Finn Chambers on Scanpor are intended for use in epicutaneous diagnostic patch testing and clinical research by or under the supervision of a trained healthcare professional only.

Contraindications:

Confirmed Type IV hypersensitivity to aluminum.

Warnings and Precautions:

- Use polypropylene coated chambers if testing with an aqueous mercuric solution.
- Do not cut strips between the chambers, as the remaining adhesive area may be too small.
- Use caution when administering uncommon test substances and selecting their test concentration.
- Apply patch test panels on healthy, clean skin only. Instruct patients to refrain from showering or excessive sweating, as the panel adherence may be compromised.
- See adverse events section for more warnings.

Special Patient Populations:

Finn Chambers on Scanpor can be used in all patients consistent with the limitations associated with the use of individual allergens and patch testing in general.

Adverse Events:

- Allergic reactions to aluminum and Scanpor tape are very rare.
 - However, occasional cases have been reported.
 - Skin may react to the removal of the tests with a slight mechanical irritation, indicated by erythema.
- Instructions for Use:**
- Identify each panel and chamber being used for the patch test (numbers).
 - Remove the protective paper backing to reveal the chambers, leaving the narrow handling strip in place for easiest application onto the patient.
 - *Semisolids: apply directly into the chamber.*
 - *Liquids: use filter paper discs to absorb the liquid being held in the chamber. Note: Do not let the paper discs dry, as this could result in a false negative reaction.*
 - Apply the liquids and volatile substances immediately prior to application on the patient.
 - Apply the panels on the patient's back, or suitable alternate location.
 - Apply the panel starting at the bottom and working upwards pressing each chamber in place and working out any air trapped between the tape and skin.
 - Remove the remaining paper-backing ensuring the entire panel is sticking to the patient's back.
 - Mark the placement of the panels on the skin.
 - At removal, each test site should have a ring-shaped depression. This verifies occlusion during the patch test.
 - Allow 20 minutes for transient erythema to dissipate, prior to reading initial results.
 - Discard the used panels in accordance with local regulations.

Serious Incidents:

Report all serious incidents:

In the USA: to the manufacturer and the FDA;

In the EC: to the European Representative, or local competent authority.

In other countries: to the distributor or local competent authorities.

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